



INSTALLATION INSTRUCTIONS

APF Tub Seat

APFSHPQ-265150PWS, APFSHPQ-265150PTS



Customer Support: 1.877.947.7769
Email: apsupport@accessibilitypro.com



APF Portable Tub Bench Installation Instructions

**ADA REGULATIONS SPECIFY A NISHED SEAT HEIGHT 17" TO 19" ABOVE THE NISHED
OOR AND 1½" OF CLEARANCE BETWEEN THE SEAT EDGE AND THE SIDE WALL.
YOUR SHPQ PORTABLE TUB BENCH SHOULD BE INSTALLED AS AN ASSEMBLED UNIT.**


PLEASE READ THESE INSTALLATION INSTRUCTIONS THOROUGHLY PRIOR TO INSTALLING
THE SEAT TO IDENTIFY THE TOOLS RECOMMENDED FOR INSTALLATION, AS WELL AS THE
MOUNTING HARDWARE REQUIRED TO COMPLETE THE INSTALLATION. IMPORTANT: THIS
PRODUCT SHOULD ONLY BE INSTALLED BY QUALIFIED AND TRAINED PERSONNEL

FAILURE TO FOLLOW THESE INSTRUCTIONS COULD LEAD TO IMPROPER INSTALLATION OF
THE TUB SEAT AND/OR SERIOUS BODILY INJURY IN USE. IN ADDITION, ALL WARRANTIES,
EXPRESS OR IMPLIED, WILL BE NULL & VOID DUE TO IMPROPER INSTALLATION OF THIS
PRODUCT. SEACHROME CORPORATION DOES NOT ASSUME ANY LIABILITY FOR DAMAGE
RESULTING FROM SERVICES PERFORMED BY OTHERS, OR FAULTY INSTALLATION, MISUSE
OR MISAPPLICATION OF GOODS.

**MAXIMUM WEIGHT CAPACITY FOR THIS PRODUCT IS 250 LBS.
WHEN PROPERLY INSTALLED.**

1. Determine desired tub bench location.

The bench must be located on the bathtub so that the swing-down legs are perpendicular to the tub floor, and that the adjustable leveling feet are positioned on a clean, flat surface.

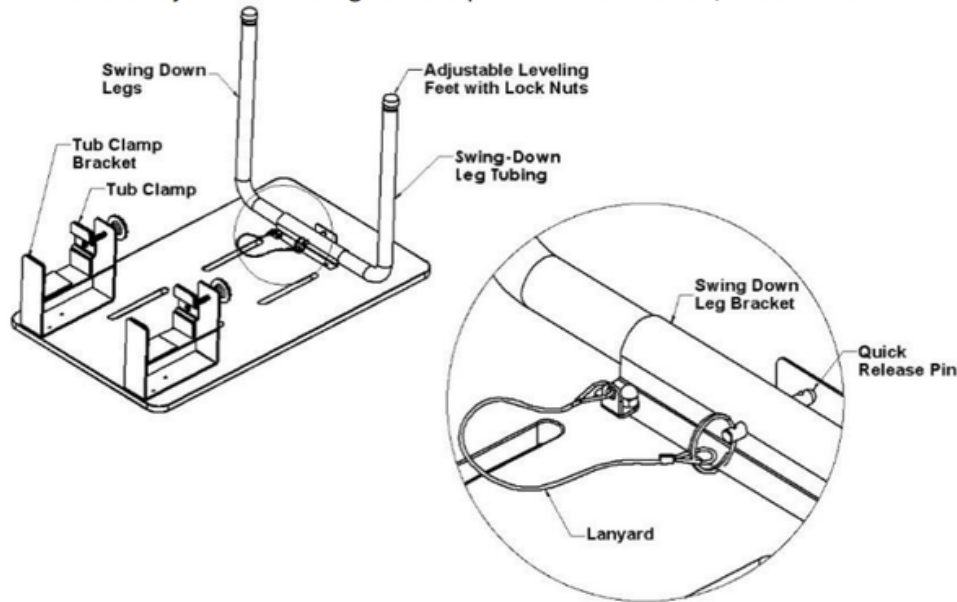

Remove the SHPQ portable tub bench from the shipping carton.


Lay the bench top on a clean flat surface so the "Tub Clamps" and "Swing Down Legs" are accessible (facing up). Manually rotate the Swing Down Legs so they are positioned perpendicular to the bench top.

Line up the holes in the Swing Down Leg Tubing with the pass-through holes in the center of the Swing Down Leg Bracket.

Push the open end of the stainless steel Quick-Release Pin through the Swing Down Leg Bracket, passing through the holes in the Swing Down Leg Tubing, until the "button" on the Quick-Release Pin has passed-through, and is completely visible, outside the opposite hole on the Swing Down Leg Bracket (see Diagram).

2. Determine seat height from the floor and location relative to the tub and bathroom finished floor.

ADA regulations specify a finished seat height of 17" to 19" above the bathroom finished floor.

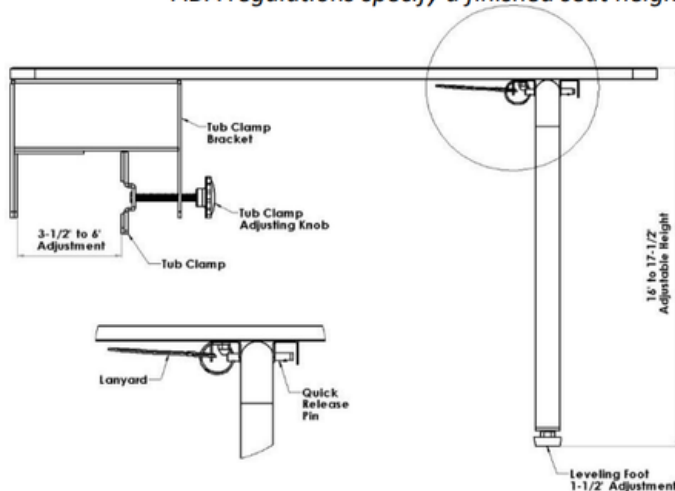

Holding the bench in its horizontal position in the tub, open the "Tub Clamps" so they fit over the tub ledge.

At this point, the length of the "Swing Down Legs" may require adjusting so the bench rests in a flat horizontal position. To change the height of the legs, adjust the length of the adjustable leveling feet and tighten the lock nut to secure them in place.

Tighten the Tub Clamps to secure the bench onto the tub ledge.

Do not "over-tighten" the Tub Clamps, which may distort the Tub Clamp Bracket Assembly.

Make certain that the leveling feet are resting on a clean, flat surface on the tub floor.

3. Check Seat for:

- 1) Correct finished height.
- 2) Tub Clamps properly secured on tub ledge.
- 3) "Swing-Down Legs" locked in down position @ 90 degrees, perpendicular to the tub floor.
- 4) The adjustable leveling feet are positioned on a clean, flat surface on the tub floor, not resting on the curved/radius corner of the tub floor.

TO ASSURE PROPER INSTALLATION:

The Swing Down Legs must be perpendicular to the tub bench and the Leveling Feet **MUST** be located on a clean **FLAT** surface on the tub floor.

Once secured to the tub, make certain the seat does not shift from its set position in any direction, before use.

MAXIMUM WEIGHT CAPACITY FOR THIS PRODUCT IS 250 LBS.

THIS COMPLETES THE INSTALLATION OF THE PORTABLE TUB SEAT.

Removing the Bench:

When removing the tub bench, reverse the above procedures:

1. Loosen (turn counter-clockwise) the Tub Clamps to remove the bench from the tub ledge.
2. Lay the bench on a clean, flat surface so the Swing Down Leg Bracket is accessible.
3. Pull the quick-release pin to remove it from the center hole of the Swing Down Leg Bracket.
4. Manually rotate the Swing Down Legs until they rest on the underside of, and horizontal to, the bench top.
5. Store this product in a clean, dry location for future use.