

This seat should be installed as an assembled unit.

PLEASE READ THESE INSTALLATION INSTRUCTIONS THOROUGHLY PRIOR TO INSTALLING THIS SHOWER SEAT


IMPORTANT: THIS PRODUCT SHOULD ONLY BE INSTALLED BY QUALIFIED AND TRAINED PERSONNEL

FAILURE TO FOLLOW THESE INSTRUCTIONS COULD LEAD TO IMPROPER INSTALLATION OF THE SHOWER SEAT, STRUCTURAL AND/OR WATER DAMAGE TO THE WALL AND IN-WALL SUPPORT STRUCTURES, AND/OR SERIOUS BODILY INJURY IN USE. IN ADDITION, ALL WARRANTIES, EXPRESS OR IMPLIED, WILL BE NULL & VOID DUE TO IMPROPER INSTALLATION OF THIS PRODUCT. THE MANUFACTURER DOES NOT ASSUME ANY LIABILITY FOR DAMAGE RESULTING FROM SERVICES PERFORMED BY OTHERS, OR FAULTY INSTALLATION, MISUSE OR MISAPPLICATION OF GOODS.

THIS PRODUCT IS DESIGNED TO MEET OR EXCEED ALL LOAD REQUIREMENTS ESTABLISHED IN THE FEDERAL SPECIFICATION #WW-P-541/8b AT 250 POUNDS OF FORCE. THE PERFORMANCE OF THIS PRODUCT IS SIGNIFICANTLY IMPACTED BY THE STRENGTH OF THE WALL TO WHICH IT IS ATTACHED.

TO ASSURE PROPER AND SAFE INSTALLATION, THIS SHOWER SEAT MUST BE INSTALLED INTO A MINIMUM 2" THICK, SOLID WOOD, IN-WALL BACKING MATERIAL. MAKE CERTAIN IN -WALL BACKING COMPLIES WITH ALL LOCAL BUILDING CODE REQUIREMENTS.


1. Determine the desired tub seat location.

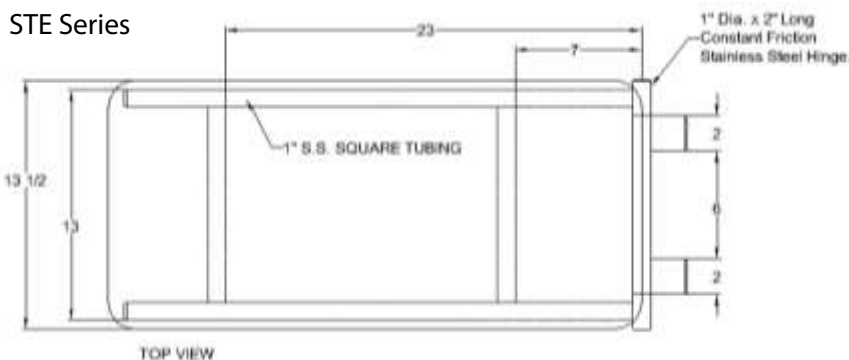

Remove the tub seat from the shipping carton.

Locate the seat in its proper position on the tub.

STE Series: Position the Tub Seat so that when mounted to the tub wall, the end opposite to the tub hinges rests securely on the Tub Dam or Apron.

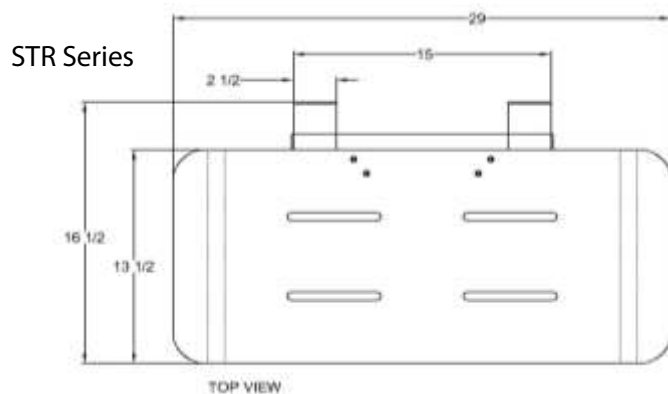

STR Series: Position the Tub Seat so that when mounted to the tub wall, the two sides adjacent to the tub hinges both rest securely on the Tub Dam or Apron.


INSTALLATION INSTRUCTIONS

END-HUNG (STE) &
REAR-HUNG (STR)
TUB SEAT

2. Mounting the tub seat to the wall.

Use six (6) #10-24 x 2½" stainless steel screws (not supplied by manufacturer) to secure mounting hinges to the wall.

STE Series

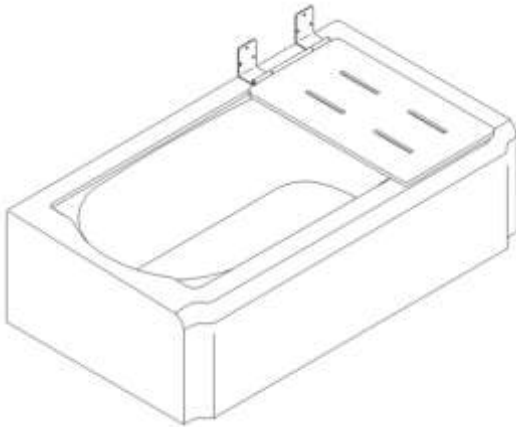


With the two hinges placed against the tub wall, mark the hole locations for each hinge. Remove the seat from the tub.

With a center punch, punch the center of each hole location.

Fasten each hinge to the wall securing and tightening each fastener.

STR Series

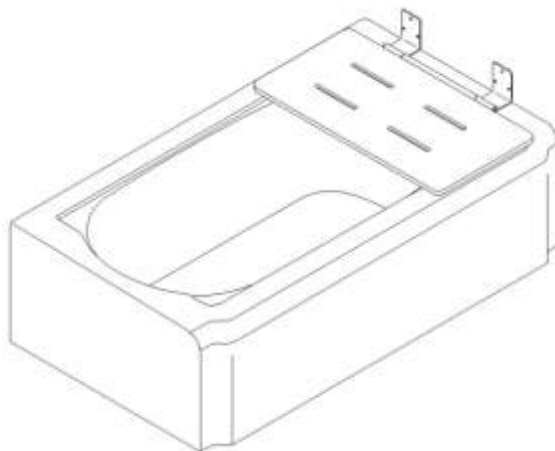

Note: If the wall of the tub is tile, stone or other hard material, a carbide tipped drill of the proper diameter should be used.

Once secured to the tub, make certain the seat does not shift from its set position in any direction before use.

THIS COMPLETES THE INSTALLATION OF THE STE or STR TUB SEAT.

Recommended tools:

Procure six (6) #10-24 x 2½" stainless steel screws, tape measure, clear tape, 1/4" power drill, 1/4" carbide tip drill (if you have tile, marble or granite surface), 1/4" plain drill (if you have acrylic or fiberglass surface), 1/8" plain drill, pen or pencil, center punch, level, hammer, phillips head screw driver.